



Disinfection Guide for Canon Eye Care Products

CR-2AF
CR-2 Plus AF
CR-2
CF-1
CX-1
TX-20
RK-F2

CANON MEDICAL SYSTEMS CORPORATION



Procedure:

The procedure of cleaning and disinfection is specified in the operation manual of each system. Please refer to the maintenance section of the operation manual. For disinfection, please refer to the list of approved solvents and wipes in the table below.

WARNING: Turn off the power and unplug the device from the AC outlet before cleaning the unit.

Applicable Eye Care Products: CR-2AF, CR-2 Plus AF, CR-2, CF-1, CX-1, TX-20, RK-F2

List of Solvents/Wipes for Eye Care Products					✔: Usable with no influence on the products.				
Classification	Name of product	Manufacturer	Region	[USA] EPA Reg No.	Outer Covers	Chin Rest / EOS	Head Rest / Objective Lens Cover	Metal (Joystick and Arm for CX-1)	Objective Lens/Finder Lens
Oxidant	Accel TB Wipes	Virox Technologies Inc	USA	74559-3	✓	✓	✓	✓	not usable
Chlorine	Sani-Cloth® Bleach Wipes	Professional Disposables International, Inc.	USA	9480-8	✓	✓	✓	not usable	not usable
Chlorine	Clorox Healthcare™ BLEACH GERMICIDAL WIPES	Clorox Professional Products Company	USA	5813-73 5813-108 67619-12?	not usable	not usable	not usable	not usable	not usable
Quarternary Ammonium Salts	INCIDIN FOAM	Ecolab Ltd.	USA	NA	✓	✓	✓	not usable	not usable
Quarternary Ammonium Salts	SANI-CLOTH AF3	Professional Disposables International, Inc.	USA	9480-9	✓	✓	✓	not usable	not usable
Quarternary Ammonium Salts	Super Sani-Cloth	Professional Disposables International, Inc.	USA	9480-4	✓	✓	✓	not usable	not usable
Quarternary Ammonium Salts	SANI-CLOTH PLUS	Professional Disposables International, Inc.	USA	9480-6	not usable	not usable	not usable	not usable	not usable
Quarternary Ammonium Salts	SANI-CLOTH HB	Professional Disposables International, Inc.	USA	61178-4-9480	✓	✓	✓	not usable	not usable
Quarternary Ammonium Salts	SANI-CLOTH AF	Professional Disposables International, Inc.	USA	9480-5	✓	✓	✓	not usable	not usable
Quarternary Ammonium Salts	CaviWipes	METREX RESEARCH CORP	USA	46781-8	not usable	not usable	not usable	not usable	not usable