



# **Disinfection Guide for 40kW Mobile FLEX PLUS Digital X-Ray System**

**CANON MEDICAL SYSTEMS CORPORATION**

## **CLEANING AND DISINFECTION**

The procedure of cleaning, and disinfection are specified in the operation manual of each system.

**WARNING:** NEVER ATTEMPT TO CLEAN ANY PART OF THE UNIT WHEN IT IS SWITCHED ON.

- 1) Clean the equipment frequently, particularly if corroding chemicals are present.
- 2) Clean external covers and surfaces, especially parts which might be in contact with patients, with a cloth moistened in warm water with mild soap. Wipe with a cloth moistened in clean water.

**WARNING:**  
DO NOT APPLY DIRECTLY ANY LIQUID ON THE SCREEN OR OTHER SURFACES,  
NOR USE CLEANERS CONTAINING BLEACH, AMMONIA OR ANY OTHER ABRASIVE  
OR SOLVENT LIQUID, IT COULD CAUSE DAMAGE TO THE EQUIPMENT.