



# **Disinfection Guide for OMNERA 400T Auto-Positioning Digital Radiographic System**

CANON MEDICAL SYSTEMS CORPORATION

## **Cleaning Recommendations**

Surfaces that are in contact with the patient shall be cleaned with a lint-free cloth and a small amount of soap and water or cleaning spirit

**(1) How to clean the collimator:**

- (a) Switch off the main power supply.
- (b) Use non-abrasive cleaning products. Care must be taken to prevent liquid from entering the collimator.

**(1) How to clean the X-ray tube:**

- (a) Surfaces that are in contact with the patient shall be cleaned with a lint-free cloth and a small amount of soap and water or cleaning spirit.

**(1) How to clean the ceiling suspended X-ray tube support:**

- (a) Surfaces that are in contact with the patient shall be cleaned with a lint-free cloth and a small amount of soap and water or cleaning spirit.

**(1) How to clean the table:**

- (a) Surfaces that are in contact with the patient shall be cleaned with a lint-free cloth and a small amount of soap and water or cleaning spirit.

**(1) How to clean the wallstand**

- (a) Surfaces that are in contact with the patient shall be cleaned with a lint-free cloth and a small amount of soap and water or cleaning spirit.

**Note:**

Use a moderate amount of liquid, when cleaning the product! The collimator is not water-resistant.