



**Disinfection Guide for
SOLTUS® 450 and
SOLTUS® 500
Mobile Digital X-Ray
Systems**

CANON MEDICAL SYSTEMS CORPORATION

Procedure

The procedure of cleaning, and disinfection are specified in the operation manual of each system.

CLEANING AND DISINFECTION NEVER ATTEMPT TO CLEAN ANY PART OF THE UNIT WHEN IT IS SWITCHED ON.

Clean the equipment frequently, particularly if corroding chemicals are present. Clean external covers and surfaces, especially parts which might be in contact with patients, with a cloth moistened in warm water with mild soap. Wipe with a cloth moistened in clean water.
When it is needed to disinfect the Control Console, clean it with a cloth impregnated with isopropyl alcohol.

**DO NOT APPLY DIRECTLY ANY LIQUID ON THE SCREEN OR
OTHER SURFACES, NOR USE CLEANERS CONTAINING
BLEACH, AMMONIA OR ANY OTHER ABRASIVE OR SOLVENT
LIQUID, IT COULD CAUSE DAMAGE TO THE EQUIPMENT.**